

DE Blutdruck-Messgerät HGH
GB Blood pressure measuring device HGH
FR Tensiomètre HGH
IT Sfigmomanometro HGH

Gerät und LCD-Anzeige
Unit and LCD Display
Appareil et Affichage LCD
Apparechio e Display LCD

2 x LR03, 1,5 V, AAA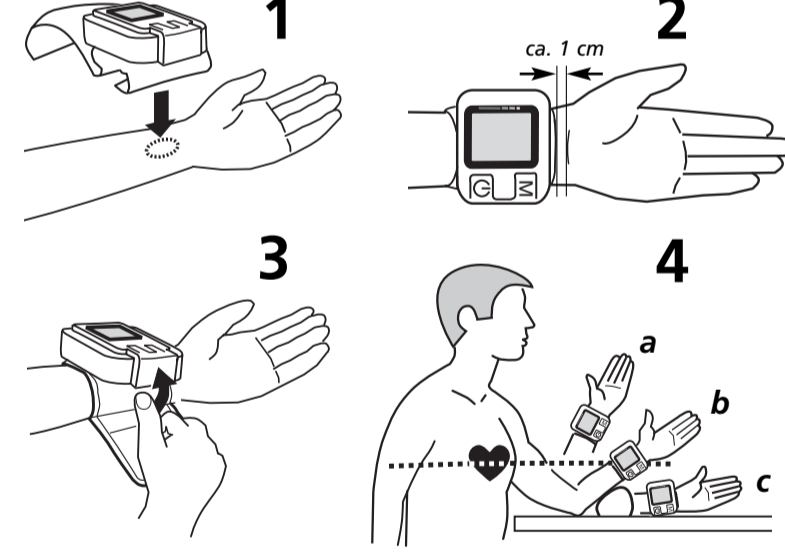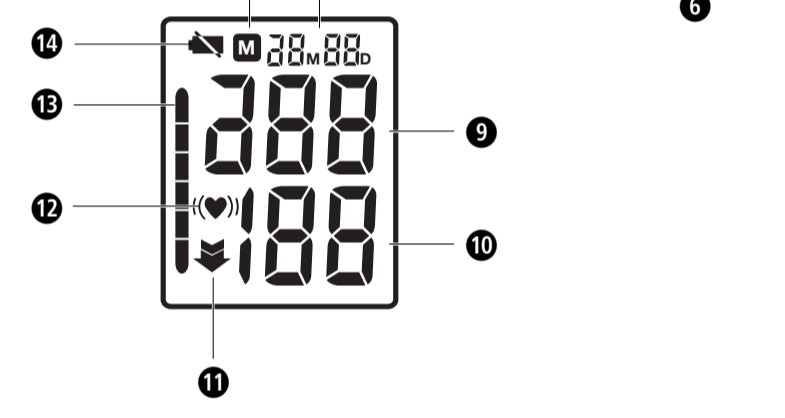

Zeichenerklärung Explanation of symbols
WICHTIG IMPORTANT IMPORTANTE

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GB Safety instructions
Read the instruction manual carefully...

Deleting the memory
Press and hold the M button...

Blood pressure classification according to the WHO
These values have been specified by the World Health Organisation...

Error messages and error remedying
1. Er (Replace both batteries)...

Cleaning and maintenance
Remove the batteries before cleaning the unit. Clean the unit using a soft cloth...

Unit and LCD display
1. Battery Compartment 2. Blood Pressure Indicator...

Scope of supply
Please check first of all that the device is complete...

Starting up
Insert/replace batteries
You must always insert the batteries provided before you can use your unit...

Guidelines / Standards
This blood pressure monitor meets the requirements of the EU standard for non-invasive blood pressure monitors...

Technical specifications
Name and model: MEDISANA Blood pressure measuring device HGH

Fitting the cuff
Place the cuff on the left wrist with the palm of the hand facing upwards...

The correct position
Carry out the measurement whilst seated. Relax your arm and place it comfortably on a table...

Taking a blood pressure measurement
1. Switch the unit on by pressing the START button...

Warranty and repair terms
Please contact your dealer or the service centre in case of a claim under the warranty...

Hersteller/Manufacturer/Fabricant/Produttore
Herstellungsdatum/Date of manufacture
Date de fabrication/Date di produzione

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